



Instructions for Use

ICELOCK[®] 272 MALE PYRAMID TI
ICELOCK[®] 273 MALE PYRAMID SS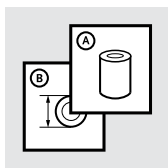

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	7
FR	Notice d'utilisation	11
ES	Instrucciones para el uso	15
IT	Istruzioni per l'uso	19
NO	Bruksanvisning	23
DA	Brugsanvisning	26
SV	Bruksanvisning	29
EL	Οδηγίες Χρήσης	32
FI	Käyttöohjeet	36
NL	Gebruiksaanwijzing	39
PT	Instruções de Utilização	43
PL	Instrukcja użytkownika	46
TR	Kullanım Talimatları	50
RU	Инструкция по использованию	53
JA	取扱説明書	57
ZH	中文说明书	60
KO	사용 설명서	63

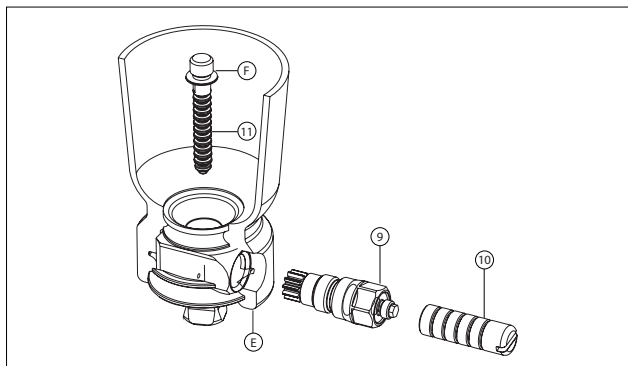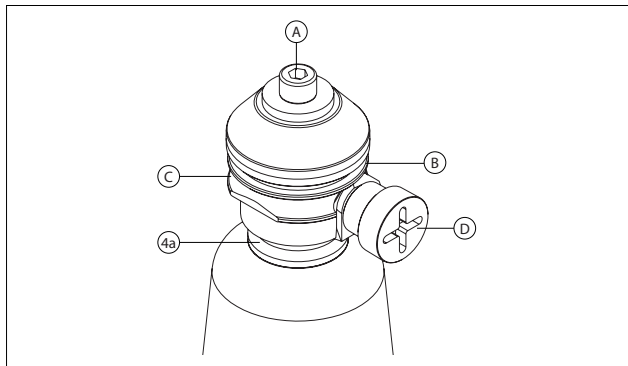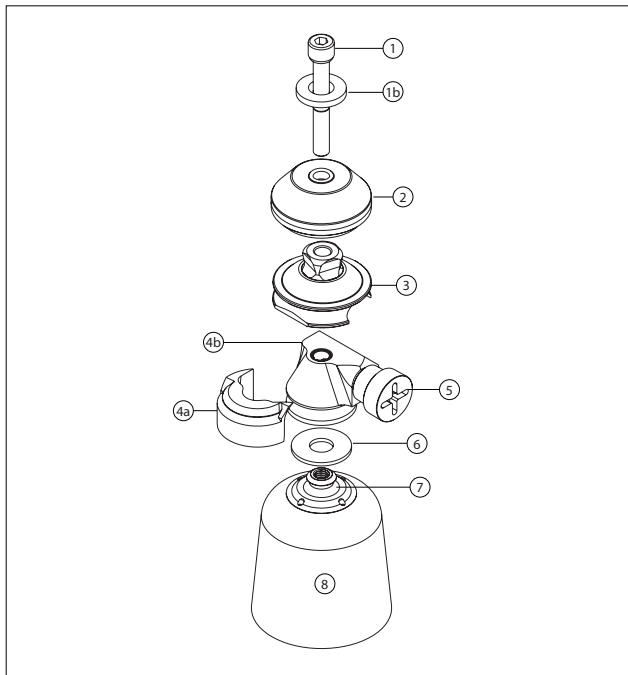Medical Device

INTENDED USE

The device is intended as a connection component of a prosthetic system that replaces a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

For more detailed device information, see **Table 1**.

Indications For Use and Target Patient Population

- Lower limb loss, amputation, or deficiency
- No known contraindications

For weight limit and impact level, see **Table 2**.

GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

The device is for single patient use.

ASSEMBLY INSTRUCTIONS

Components:

- Stainless Steel Pyramid w/ O-Ring (3).
- Pyramid Cap (2).
- Plastic Washer M8 (1b)
- Pyramid Bolt Long w/ O-Ring (1).
- Silicone Fill for 273 Pyramid (4a).

OR

- Titanium Pyramid w/ Spacer & O-Ring (3).
- Pyramid Cap (2).
- Plastic Washer M8 (1b)
- Pyramid Bolt Long w/ O-Ring (1).
- Silicone Fill for 272 Pyramid (4a).

Use with L-211000 Icelock Clutch 211

Lamination Instructions:

1. Flatten distal end of Plaster Cast (8) to center Delrin Guide (7). Attach with nails provided or double-sided tape maintaining original pin alignment.
2. Apply inner PVA bag and tie it around 'neck' on Delrin Guide.
3. Trim bag around string and place Silicone Seal (6) over 'neck' of Delrin Guide (7), onto PVA bag. Bond Silicone Fill (4a) to Lock Body (4b) using double-sided tape.
4. Place Lock Body (4b) and connected Silicone Fill (4a) onto Delrin Guide (7), making sure the Dummy Screw (5) for Clutch Mechanism (9) is correctly positioned.
5. Place Pyramid (3) onto Lock Body (4b), making sure that O-Ring in groove on pyramid is in place.

6. Put Pyramid Cap (2) and Plastic Washer (1b) onto Pyramid (3) and secure them with Pyramid Bolt (1) (make sure that O-Ring on Bolt is in place).
7. Tighten the Bolt (1).
Note: Be sure to tighten (1-2 Nm) so that resin will not leak in between components.
8. Apply clay into hexagon hole (A) on bolt and groove (D) on Dummy Screw.
9. Tie lay-up material around Lock Body (4b) and Silicone Fill (4a) and into groove (C) on Pyramid and groove (B) on Delrin Cap.
Warning: Tie at least two layers of fibers in groove (B) on Delrin Cap.
10. Apply outer PVA bag and pour resin.
11. When curing has finished, grind down to Pyramid Bolt (1) and Dummy Screw (5) and remove.
12. Remove Pyramid Cap (2).
Warning: Do not grind material away from distal end of socket (E) since that reduces strength of connection between socket and pyramid.
13. Remove socket from plaster cast.
14. Insert and tighten Clutch Mechanism (9) to 15 Nm.
Warning: Make sure to tighten the Clutch Mechanism to 15 Nm.
Note: Push Button (10) may be shortened by cutting it with hacksaw.

Attachment Pin:

Apply Loctite 410/411 onto threads (F) on Attachment Pin (11) and screw it into Locking Liner. Torque Attachment Pin to 4 Nm. Put Protector Cap on Pin before donning liner to prevent damage. Remove Cap before donning socket. Also available:

- L-292003 Attachment Pin clutch, short w/cap.
- L-292005 Attachment Pin clutch, long w/cap.

Caution: Use attachment pins for Icelock 200 series only.

T-wrench supplied is specially designed to wind Push Button (10) on Clutch Mechanism (9) when donning socket.

Caution: Use of any other tool may result in over tightening.

USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

Environmental Conditions

See which environmental conditions apply to the device in **Table 1**.

A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e.g., rain), no submersion is allowed.

No contact with salt water or chlorinated water is allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

A Waterproof device can be used in a wet or humid environment and submerged in up to 3-meter-deep water for a maximum of 1 hour.

It can tolerate contact with: Salt water, chlorinated water, perspiration, urine, and mild soaps.

It can also tolerate occasional exposure to sand, dust, and dirt.

Continuous exposure is not allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water after exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

MAINTENANCE

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

SPECIFICATIONS

Table 1 - Device Specifications

Item Number	Item Description	Environmental Conditions
L-273000	Icelock Stainless Steel Pyramid 273	Weatherproof
L-272000	Icelock Titanium Pyramid 272	Waterproof

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Table 2 - Usage Specifications

Item Number	Impact Level	Weight Limit	ISO 10328 - Label Text
L-273000	The device is for low to moderate impact use, e.g., walking.	100 kg	ISO 10328 - P5 - 100 kg
L-272000	The device is for low to high impact use, e.g., walking and occasional running.	166 kg	ISO 10328 - P7 - 166 kg

ISO 10328 - "P" - "m"kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

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