ENGLISH

MD

Medical Device

DESCRIPTION

The device is an auto-expulsion valve with push button for suction release.

Components (Fig. 1)

- A. Valve
- B. Membrane
- C. Retainer
- D. O-ring
- E. Housing
- F. 2x Nails
- G. Dummy for Flexible Sockets (WHITE or GREEN)
- H. Dummy for Rigid Sockets (RED)

INTENDED USE

The device is intended to connect and release a prosthetic system that replaces a missing lower limb. Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional. The device must be fitted and adjusted by a healthcare professional.

Indications For Use and Target Patient Population

- Lower limb loss, amputation, or deficiency
- No known contraindications

The device is for high impact use, e.g., walking and occasional running.

GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

The device is for single patient use.

ASSEMBLY INSTRUCTIONS

Cast rectification
1. Determine the Valve location on the plaster model and flatten the area to ensure the Dummy sits flush. In the center hole of the Dummy place a mark on the plaster model with an indelible marker.

2. Drill a 2 mm vacuum hole using a long drill.

3. Attach the Dummy to the plaster model, using the Nails provided.

Caution: For rigid socket choose RED dummy. For flexible inner sockets choose WHITE or GREEN dummy.

Preparation for thermoplastic sockets
1. Drape the thermoplastic socket.
2. Guide the plastic around dummy to ensure proper molding.

Preparation for laminated sockets
1. Place the first PVA bag over the plaster model with dummy in place.
2. Place the reinforcing layers and second PVA bag over the

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Reassembly

Replace the Membrane with convex surface facing down (Fig. 2).

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIAZTILITY

OÙSS does not assume liability for the following:

• Device not maintained as instructed by the instructions for use.

• Device assembled with components from other manufacturers.

• Device used outside of recommended use condition, application, or environment.

Preparation for laminated sockets

1. Place the first PVA bag over the plaster model with dummy in place.

2. Place the reinforcing layers and second PVA bag over the

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