Rev. 3 - 04/12_100161

Material
Polyamid (PA)
Polyurethan (PUR)
Edelstahl
Polyoxmethylen (POM)
Elastodien (ED)
Polypropylen (PP)
Baumwolle (CO)
10.9 galZn
Polyethylenterephalat
Nitrilkautschuk
Elastan (EL)
X 10 Cr Ni S 18 9
Polyester (PES)

Material
Polyamide (PA)
Polyurethane (PUR)
Stainless steel
Polyoxmethylene (POM)
Elastodiene (ED)
Polypropylene (PP)
Cotton (CO)
10.9 galZn
Polyethylenterephalate
Nitrile rubber
Elastane (EL)
X 10 Cr Ni S 18 9
Polyester (PES)

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Stand der Information: November 2010

EN Dear patient,

Thank you for choosing a product from Bauerfeind® Orthopedics. SofTec® Dorso is a high-quality medical product. Please read the instructions carefully and if you have any questions contact your doctor, medical retailer or our Technical Service department.

SofTec® Dorso is an orthosis¹ for straightening and stabilizing the spinal column.

⚠ SofTec® Dorso must be individually adapted to the patient by a trained orthopaedic technician in order to gain the maximum effect and optimal wear comfort from this orthosis.
If the product is not used properly any claim for warranty will expire.

Indications

- Non-surgical treatment of vertebral fractures
- Osteoporosis
- Postoperative stabilization
- Stabilization of tumors or inflammatory processes (spondylitis)

Side effects

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

Contra indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

- 1 Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- 2 Impaired sensation.
- 3 Severe restriction of the performance of heart and lungs (risk of a rise in blood pressure if the aid is applied during severe physical exertion).

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

Fitting and removing SofTec® Dorso

Fitting:

- Before fitting, the product is adapted to the patient by the orthopedic technician.
- Put on the orthosis like a backpack, passing your arms through the shoulder straps.
- Ensure that the orthosis is correctly positioned (Fig. 1)!
- Place the fingers in the specially provided finger pockets on the two broad, light-grey fastener halves in the abdominal area and pull fastener forward uniformly.
- Press the left half of the fastener against the abdomen and pull the right half over it to secure the Velcro® fastening (Fig. 2).
- When securing the Velcro® fastening, remove the fingers from the left pocket first, followed by the right (Fig. 3).
- Take the two lower LPT (Low Power Tension) straps in both hands and pull forward at the same time to produce the correct level of force and secure both LPT straps against the large Velcro® fastening. Repeat this process for the upper LPT tension straps (Fig. 4).
- To achieve the desired correction / stability, place the thumbs in the specially provided red loops on the pre-adjusted shoulder straps and pull the straps forward and down (Fig. 5). When the upper body is straightened as required, secure the ends of the straps against the abdominal Velcro® fastening (Fig. 6).
- If you notice an unpleasant sensation of pressure below the shoulder strap, the accompanying pads can be fitted to the shoulder straps to absorb the pressure.
- **Check the correct positioning of SofTec® Dorso:** The orthosis must be positioned as low as possible without pressing unpleasantly against the groin area while seated upright.
- **During the first fitting of the orthosis, the optimal, individual fit must be checked, and if necessary corrected, by a trained specialist. Correct fitting should be practiced with the patient.**

Removal:

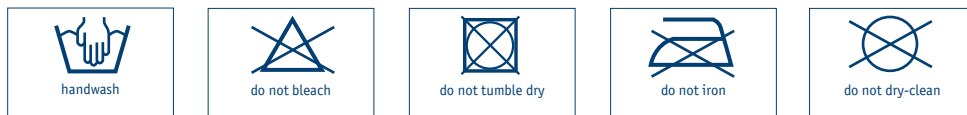
- Unfasten the shoulder straps from the abdominal fastening and refasten further up on the shoulder strap.
- Unfasten the LPT tension straps (first the upper followed by the lower straps) and then refasten without any tension.
- Open the large, light-grey fastener.
- Remove the orthosis and refasten the light-grey fastener.
- For easier handling of SofTec® Dorso, always fasten any opened fasteners directly on the product.

Important information

- SofTec® Dorso is a prescribed product that should be applied under a physician's guidance. SofTec® Dorso should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- SofTec® Dorso is intended for single use by the patient.
- The product should be applied over a T-shirt or an undershirt.
- The CFR structure (CFR = carbon fiber reinforced plastic) of SofTec® Dorso may only be adjusted by suitably trained specialists. Failure to comply with this requirement may adversely affect product performance, thereby cancelling any product liability.
- Do not make any improper modifications to the product, including subsequent modifications to the knitted section. Failure to comply with this requirement may adversely affect product performance, thereby cancelling any product liability.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.

- Please consult your doctor before using SofTec® Dorso in combination with other products.
- Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.
- SofTec® Dorso can readily be disposed of with the domestic refuse.

Product care

⚠ Never expose the orthosis to direct heat (e.g. heater, sunlight, etc.)!

- We recommend washing SofTec® Dorso separately by hand with a mild detergent at 30°C (86°F). The CFR frame must be removed prior to washing.
- The CFR cross in the back section must be unscrewed and the CFR frame pulled upwards out of the knitted section.
- To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro® fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).
- The CFR bridging frame can easily be cleaned with a sponge using a pH-neutral soap or a shampoo.
- After washing, ensure that the CFR frame, cross and shoulder straps are correctly re-inserted.

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

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